

# Breast Massage Model Type II

LM-024

## Instruction Manual

Thank you for purchasing the Breast Massage Model Type II.

Please read this instruction manual carefully to ensure correct use of the product,  
and store it in a safe place for easy access.

KOKEN CO., LTD.

1-4-14 Koraku, Bunkyo-ku, Tokyo 112-0004 JAPAN

URL:<http://www.kokenmpc.co.jp>

E-mail:[customer@kokenmpc.co.jp](mailto:customer@kokenmpc.co.jp)






## Handling and Safety Precautions

**These precautions should be strictly observed in order to ensure safe, long-term use of the product.**

### The following precautions should be strictly observed:

1. The model is made of a special silicone rubber to simulate the texture of the human skin. The model should be handled carefully, in the same manner as an actual person.
2. The surface of this model may be damaged if the silicone rubber parts are pressed or rubbed by a hard material.
3. The model may not restore to its original shape if too strongly pressed or pulled, and it may be damaged by a sharp-edged instrument or fingernail.
4. After use, or if the surface of the model feels sticky, apply the enclosed baby powder over the surface.
5. Do not draw on the model with an oil-based ink or paint.
6. Wipe off dirt or smudges on the surface using a neutral solvent, allow to dry and apply baby powder.
7. Do not block the four air holes during the procedure; otherwise, the air passage will be blocked and the surface will become wrinkled.

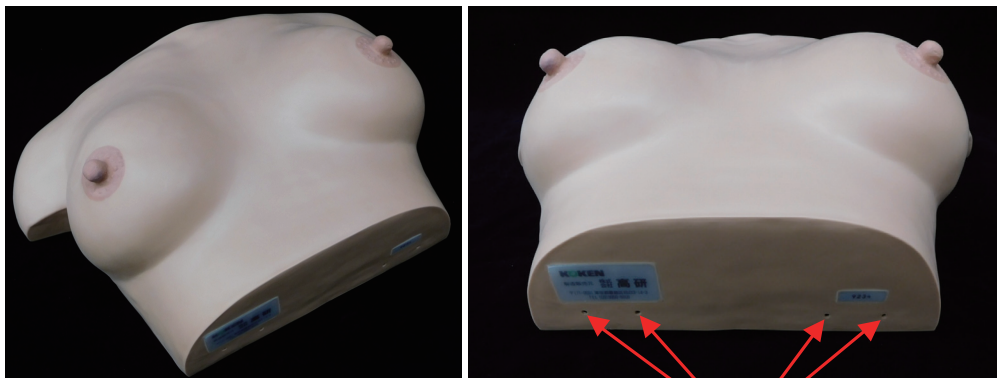

Do not block the four air holes.

1. The contents of this instruction manual are subject to change without notice.
2. This instruction manual may not be reproduced in part or in its entirety without permission.
3. Please contact the manufacturer in the event that any errors or omissions are found in the contents of this instruction manual.
4. This product should be used only as described in this instruction manual. In particular, the product should not be used in any way that contravenes the precautions noted in the instruction manual.

## 1. Outline and Features

### Outline

Flexible silicone breast massage model suitable for breast management training.

This model enables practice of detachment of the breast fundus, massage of the areola and nipple, and the procedure of milking.

### Features

1. The model is made of a special silicone rubber and the appearance and texture simulates human skin.
2. The tensity of the breast and the flexibility of the breast fundus of the puerpera are created with the utmost accuracy.
3. This model enables practice of massage of the fundus of the breast, massage of the areola and nipple, and the procedure of milking.
4. The detachment state of the breast fundus can be palpated during massage of the fundus.

## 2. Specifications

Product name	Length	Width	Height	Weight
Main Body	Approx. 29 cm	Approx. 42 cm	Approx. 13 cm	Approx. 2 kg
Storage case	Approx. 36 cm	Approx. 48 cm	Approx. 18 cm	Approx. 3.4 kg

## 3. Accessories

Products	Quantity
Main body	1
Baby powder	1
Storage case	1

## 4. Repair

In case of damage, please apply for repair by contacting our sales distributor or us.